


CARECALL4 CALL POINT FOR DISCREET DURESS

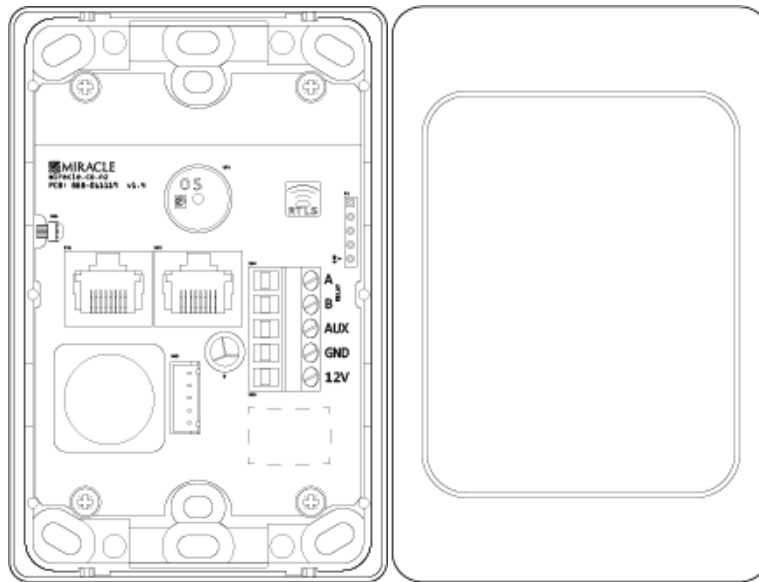


C4-CPCL-M-XX-XX
CareCall4 Call Point, No Buttons

OVERVIEW

CareCall4 Call Points on Clipsal 2000 blank plates can be provisioned for duress or discrete alarm activation. Any switch can be wired to the AUX and GND terminals of the call point.

Activated alarms must be cancelled from a CareCall EDGE terminal. Optional incident reporting can be configured which must be filled out by the user upon terminal accessed alarm resets.

WIRING

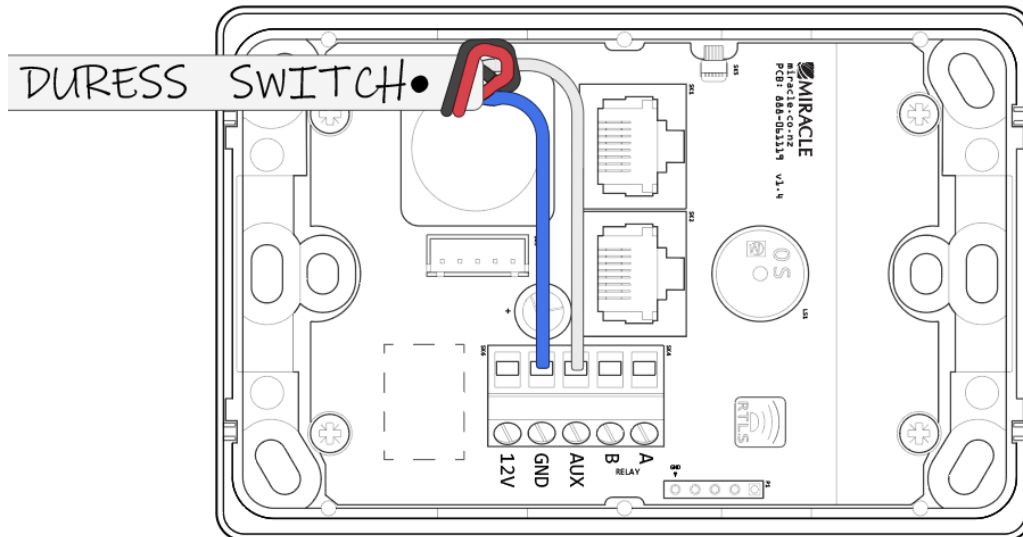

It is recommended that switch circuits are wired normally closed, though not required. 0.44mm² 4-core or similar cabling.

CARECALL4 CONFIG TOOL

The following options are the minimum configuration requirements. Additional or alternative configuration may be required depending on the specific solution. Leave all settings default unless otherwise instructed. Consult Miracle Electronics if using RTLS.

INPUT SETUP

CC4 Master Call Point

CopyPaste

Description

Serial Number

Bus Address

Call Point Address: 21

DAEADA03

21

Set

Reboot

Select

Linked Call Points: 102, 103

Input Setup

Features

Display

Back Light

Offline Alarms

RTLS

Big Button

☐ Press
 ☐ 3 x Press
 ☐ Hold
 ☐ Release

Emergency

Emergency

Wet Area

Door

☐ Button Follow
 ☐ Invert Input
 ☐ Relay
 ☐ Buzzer

Socket

☐ Press
 ☐ 3 x Press
 ☐ Hold
 ☐ Release

Patient Call

Emergency

Wet Area

Door

☐ Button Follow
 ☐ Invert Input
 ☐ Relay
 ☐ Buzzer

Pull Out

☐ Press
 ☐ 3 x Press
 ☐ Hold
 ☐ Release

Patient Call

Emergency

Wet Area

Door

☐ Button Follow
 ☐ Invert Input
 ☐ Relay
 ☐ Buzzer

Auxillary

☒ Press
 ☐ 3 x Press
 ☐ Hold
 ☐ Release

Emergency

Emergency

Emergency

Emergency

☐ Button Follow
 ☒ Invert Input
 ☐ Relay
 ☐ Buzzer

GetSet

Leave off for N/O switches.

FEATURES

CC4 Master Call Point

CopyPaste

Description

Serial Number

Bus Address

Call Point Address: 21

DAEADA03

21

Set

Reboot

Select

Linked Call Points: 102, 103

Input Setup

Features

Display

Back Light

Offline Alarms

RTLS

☐ Nurse Presence Can Cancel Patient Call
 ☐ Upgrade Patient Call To Staff Assist If Nurse Presence Is Active
 ☐ Hold Down Requires Both Call and Cancel
 ☐ Nurse Presence Canceled By Cancel Button
 ☒ Cancel LED Does Not Flash
 ☐ Feature Bit 5
 ☐ Feature Bit 6
 ☐ Feature Bit 7

Three Push Upgrade Detection Time (x 100ms)

◀

30

▶

One Button Press And Hold Detection Time (x 100ms)

◀

30

▶

Press And Hold Call And Cancel Detection Time (x 100ms)

◀

30

▶

Relay Output Setup

Disabled

◿

☐ Relay Normally Closed

☐ Enable Buzzer

Get

Set

BACKLIGHTING

CC4 Master Call Point

Copy Paste

Description

Serial Number

Bus Address

Call Point Address: 21

DAEADA03

21

Set

Reboot

Select

Linked Call Points: 102, 103

Input Setup

Features

Display

Back Light

Offline Alarms

RTLS

Standby Call LED Brightness (0 to 10)

◀ 0 ▶

Active Call LED Brightness (0 to 10)

◀ 0 ▶

Standby Cancel LED Brightness (0 to 10)

◀ 0 ▶

Active Cancel LED Brightness (0 to 10)

◀ 0 ▶

Get

Set

EDGE CONFIGURATION

RECONFIGURE PRIORITY

Inputs / Call Points

Show

☐ Inputs ☐ Call Points ☒ Call Point Groups

Call Point Groups

- System
- Melbourne Demo
- Duress
 - Duress Point 1
 - Emergency
 - Staff Assist
 - Wet Area
 - Patient Call
 - Door Alarm
 - Staff Presence
 - Self-Test Fail
 - Check In Fail
 - Duress Point 2
 - Duress Point 3

<- Add

<- Delete

<- Search

<- Copy

<- Paste

Properties

Input Name: Emergency Input Index: 155

Input Identifier:

Priority Level: Emergency : 255 Disable

☒ Include in Client Database ☐ Locally Latched

☐ Locally Resettable ☐ Activate Input (Test)

Activation Resets this Call Point: None

Location ID must be in: None

Messages and Displays

Emergency: Duress Point 1 -	Long	On Activate
Emergency: Duress Point 1 -	Smart Device	On Activate
Duress Point 1	Long	On Activate
	Pager	On Activate

Batch Modify

Delete Add New Apply

CONFIGURE TERMINAL RESET

Properties

Input Name: Duress Input Index: 155

Input Identifier:

Priority Level: Duress : 255 Disable

☒ Include in Client Database ☒ Locally Latched

☒ Locally Resettable ☐ Activate Input (Test)

Activation Resets this Call Point: None

Location ID must be in: None

Messages and Displays


INCIDENT REPORTING

Priority

Priority Name:

Duress

Icon:



>

Colour:

☒ Blue

▼

Priority Level:

255

Resend Interval:

1.00


Secondary Escalation:

0.00

Backup Escalation:


0.00

Tone:

 1


▼

Reset Tone:

 0

▼

PC Sound:

 Alarm 2

▼

DB Logging:

Yes ▼

Incident Report:

Yes ▼

Contact ID Event Code:

120

Device Assignment

☒

Vacation Disables Alarms


☐

Spectralink Icon:

0

Save

DISABLE SELF-TESTING

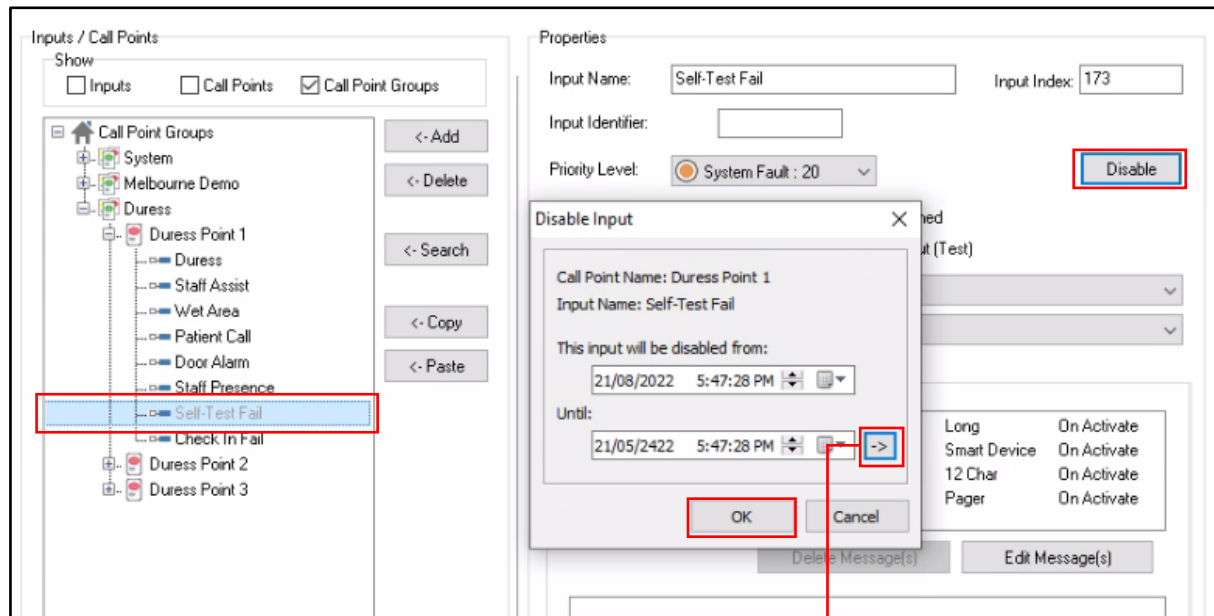

Click >3 times.