

Innovative

Quality

Leading the way in Security and HealthCare.

Integration

Hand Programming Guide

7950 Series



1. THE 7950 PAGER

1.1 Case Layout



Escape	Left	Right	Read
Key	Key	Key	Key

1.2 Hand Programming Menu

- With the pager turned on, hold down  the Escape Key and  Read Key for 3 seconds.
- You will be prompted for a four digit numeric password.

PASSWORD:0000

- Use the  Escape Key to increase the selected digit and the  Read Key to decrease the selected digit. Press the  Arrow Key to move along to the next digit. After the correct digits are entered, press the  Arrow Key to proceed to the first screen. By default this password is 0200.

- Once you are in the programming menu, you will see the first four cap codes ready for viewing or editing. There are eight cap codes available in total. The A, B, C, D, E, F, G, H at the start of the line corresponds to cap code 1 to 8.

```
A: [ ON] 0000008
B: [ ON] 0000016
C: [ ON] 0000024
D: [ ON] 0000032
```

NOTE: If you just need to change frequency or baud rate and not any capcodes, you can press the Arrow Key to jump to that screen.

- Each cap code can be enabled or disabled if required. This is what the ON or OFF field before each cap code refer to.

- Use the  Escape Key and  Read Key to change the selected field. Use the  Arrow Key to move to the next field.

- Once you have scrolled through all of the fields on this first screen, you will be taken to the second screen which contains the next four cap codes.

E: [ON]	0000008
F: [ON]	0000016
G: [ON]	0000024
H: [ON]	0000032

- Once you have scrolled through all of the fields on this second screen, you will be taken to the third screen which contains settings for the pagers baud rate, polarity and frequency.

BAUDRATE: 512 BPS	
POLARITY: NORMAL	
FREQ: 148.3375 M	
PROGRAM	EXIT

- Use the  Escape Key and  Read Key to change the selected field. Use the  Arrow Key to move to the next field. These settings must match that of your paging system.

- To save any changes you have made, move the cursor to the "Program" field and press the  Read Key. The screen will display EEPROM PROGRAMMING followed by **FINISHED**. The pager will then resume normal operation with the newly programmed settings.

EEPROM PROGRAMMING

- To exit the programming menu without saving any changes, move the cursor to the "Exit" field and press the  Read Key. The pager will then resume normal operation.

Quality • Innovative • Integration

Leading the way in Security and HealthCare.



2/27 Bate Close, Pakenham
Victoria, Australia 3782

P/ 1300 669 507
mail@ktr.com.au

F/ 1300 669 508
www.ktr.com.au